
SomnoSwiss[®] 
besser schlafen

AVA

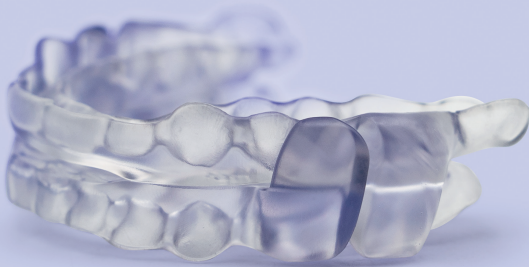Instructions for use

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Congratulations on your SomnoSwiss® AVA Sleep Appliance

This professional sleep appliance is customized to your individual dental situation by our experts and manufactured under the highest standards for medical devices. In this guide, you will find useful information and our recommendations for the correct use and care of the SomnoSwiss® AVA.

This therapy helps to improve your general health through better breathing during sleep.

your SomnoSwiss® Team - sleeping better

Indication

SomnoSwiss® AVA is a mandibular advancement appliance intended for patients with obstructive sleep apnea (OSA) in a wide range of severity, and those for whom alternative treatments are inadequate. The splint is also suitable for snoring patients and bruxism patients.

Contraindication:

The therapy should not be used


- for central sleep apnea (CSA)
- for severe breathing disorders that are not directly related to sleep apnea
- for dysfunction of the temporomandibular joint
- for loose teeth
- for advanced periodontal disease
- under the age of 18

Please note:

You must bring your SomnoSwiss® AVA to your dental appointments every time. The splint must be checked by the dentist and adjusted, if necessary, when new teeth are fitted.

1 SomnoSwiss® AVA

SomnoSwiss® AVA consists of two individual splints that together hold the lower jaw in a defined therapeutic position (protrusion). This protrusion of the lower jaw, defined by the dentist, keeps the airways open.

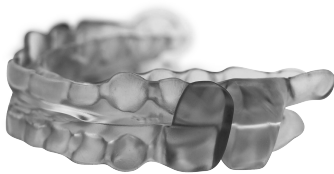AVA sleeping bite

2 Inserting and removing the splint

Before inserting the splint into the mouth, join the splints together. Place the splints in the mouth and insert the upper jaw first. Then push the lower jaw forward and bite together carefully until both splints are firmly seated on the dental arches.

To remove the splint, hold the lower or upper jaw splint firmly with both fingers and pull it off the teeth. Repeat this procedure for the other splint.

3 Acclimation Phase

The acclimation period may take up to six weeks. Wearing-comfort will improve after the familiarization period. Wearing the splint may lead to temporary but harmless muscle tension (“sore muscles”).

In the morning, your teeth may not bite together as usual due to the stretched chewing muscles. In this case, bite carefully and repeatedly on the Bite Trainer along the entire dental arch, and especially on the molars after removing the splints. This relaxes the muscles so that the teeth can bite together again as usual.


You can also perform jaw muscle training to relax the jaw muscles in accordance with the Society for Dental Sleep Medicine’s guidelines found on our website.

4 Adjustment Phase (Titration Phase)

The lower jaw is checked by the dentist during the titration phase, and, if necessary, adjustments are made.

The practitioner can order additional splints from SomnoSwiss®. Within the first year, up to five additional mandibular splints are included in the price.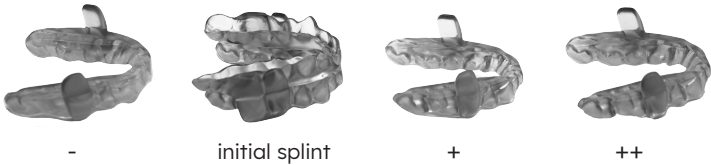

initial splint and additional mandibular splints

5 Cleaning instructions and care

SomnoSwiss® recommends cleaning the splint daily before and after wearing the SomnoSwiss AVA. Saliva and residues on the splint must be cleaned immediately after removal with cold water, a little detergent, and a soft bristle brush.

Make sure that your splint is completely dry before storing it in the tray. This is the only way to prevent the formation of bacteria.


Non-invasive dental cleaning agents can be used if necessary..

We recommend using the antibacterial CINNA Care Solution with natural ingredients from Bussmann Orthodontie-Labor AG.

CINNA Care Solution cleaning

1.  Fill a glass with water so tha the splints are covered.

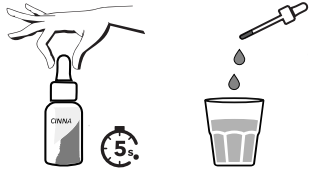2.  Press and release the pipette, wait 5 seconds until the solution rises to the top. Empty the pipette into the water and stir. Mixing ratio: 1x pipette (approx. 2ml) : 50 ml water

3.  Immerse the appliance for 30 minutes in the CINNA solution for cleaning and care of the splint.

It is recommended to repeat the procedure 1-2 times a week.

Important notes

The splints must NEVER be left in a cleaning fluid or water for longer than 30 minutes. Even in water, irreversible damage can be caused to the splint.

The splints must not be exposed to sunlight and heat.

Never use hot water to clean the appliance.

Do NOT use cleaning agents with abrasive particles (e.g. toothpaste), as this could affect the life of your splint.

Amalgam fillings may also cause dark discoloration on the splint.

6 Precautions and warnings

The use of the AVA splint may cause side effects in rare cases, which may disappear after the first few weeks of treatment or only after discontinuation of therapy. In the event these side effects persist, please contact your practitioner, who should consider the patient's medical conditions (including asthma, respiratory disorders or diseases, and other outstanding health issues) and, if necessary, refer the patient to a specialist before starting splint therapy.

Warnings

Temporary pain in the jaw muscles and temporomandibular joint

Impaired mouth breathing due to the splint


Sensitivity of teeth and gums

Increased salivation or dry mouth

Muscular bite changes that disappear after using the Bite Trainer

7 Warranty

Bussmann Orthodontie-Labor AG guarantees that the SomnoSwiss® AVA splint is free of defects from the date of delivery to the dentist. The guarantee period is 2 years. SomnoSwiss® does not guarantee the effectiveness of the AVA splint in the treatment of snoring and sleep apnea.

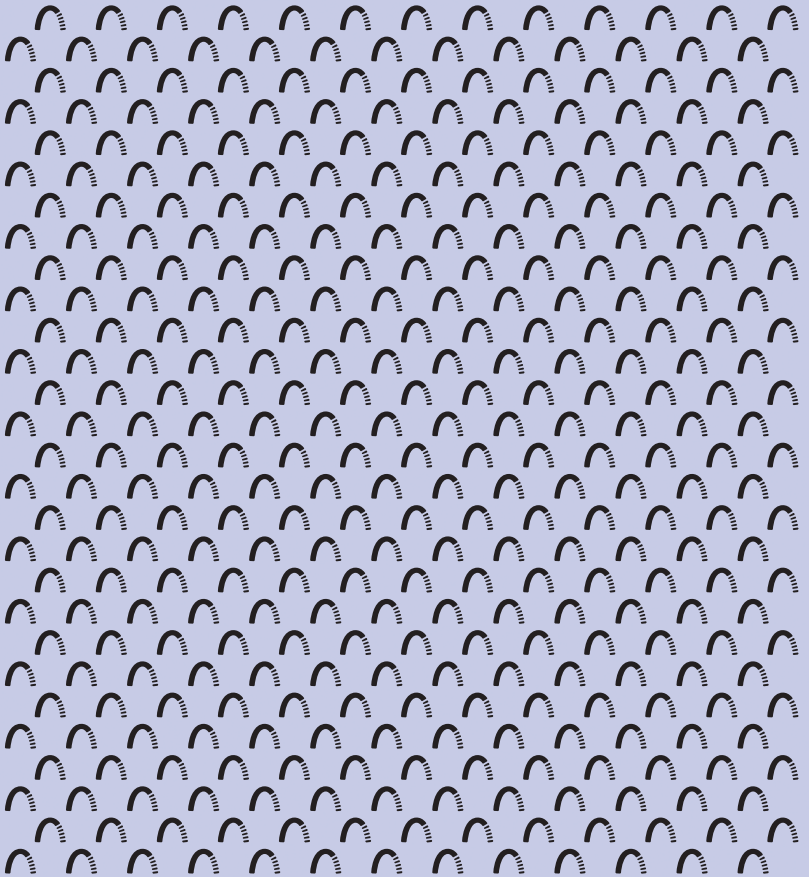Bussmann Orthodontie-Labor AG
Frankenstrasse 7a
6003 Luzern
T 041 210 00 03
info@obu.ch

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